

**NOAS**  
SWEDEN®



# Installation Instructions

## Surgery Sink- 01/03



### Installation of single and double.

If the wall cladding is not intended for loads, insert the shorts. CC between upper and lower rule should be 815 mm.


Start from which height the finished sink should sit on. Add 578 mm, mark the height.

Example: If the sink is to be 900 mm above the floor, the center hole should be drilled at 1478 mm above the floor.


Unscrew the M&-screws and pull the bracket of the sink.



Measure the center sideways and select. Drill the appropriate hole and hang the bracket with a screw.

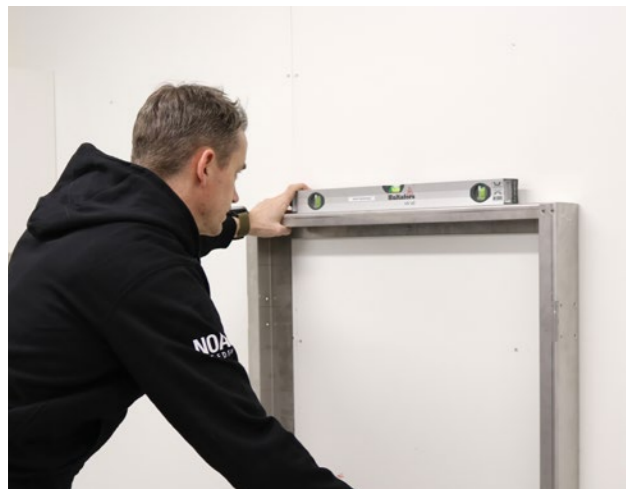

Wave the console and mark the other five holes.

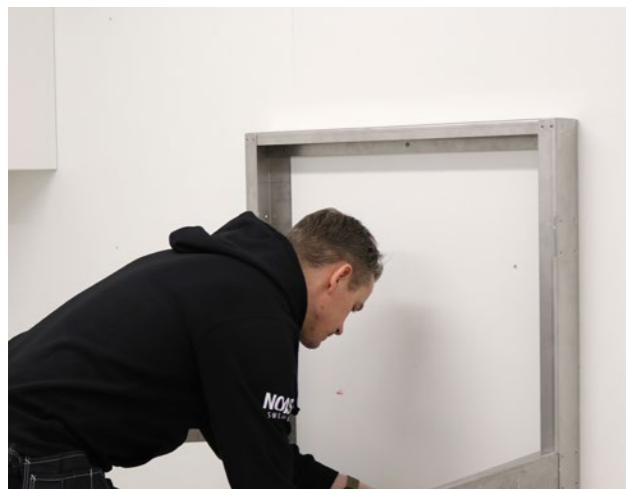

Drill and screw all screws.

NB! In case of double operation, measure the gap between the consoles so the measurement is exact. Then put the next console on the wall as above with the exact gap between the consoles.



Slide the sink onto the consoles.



Make sure the brackets are hooked correctly into the sliding rails.



Now you can install mixers, drains, soap dispensers, etc. Remember to always drill holes and use through screws with washer and nut when installing accessories in the Corian disc. Never screw in a screw or a wood screw in the Corian.

Then slide the sink fully in.



Screw the sink into the bracket with the M6 screw supplied

When everything is connected underneath on operation sink-03, the front of the front is mounted on with the screws that are included.



Other information:

Cranes are recommended to be mounted at the center of the height of the back piece. Holes are made before setting up the surgical sink.



Pipes and wires are easily hidden behind in the hollow thick back.

Metrics that are often requested are shown below.

### Surgery Sink - 01

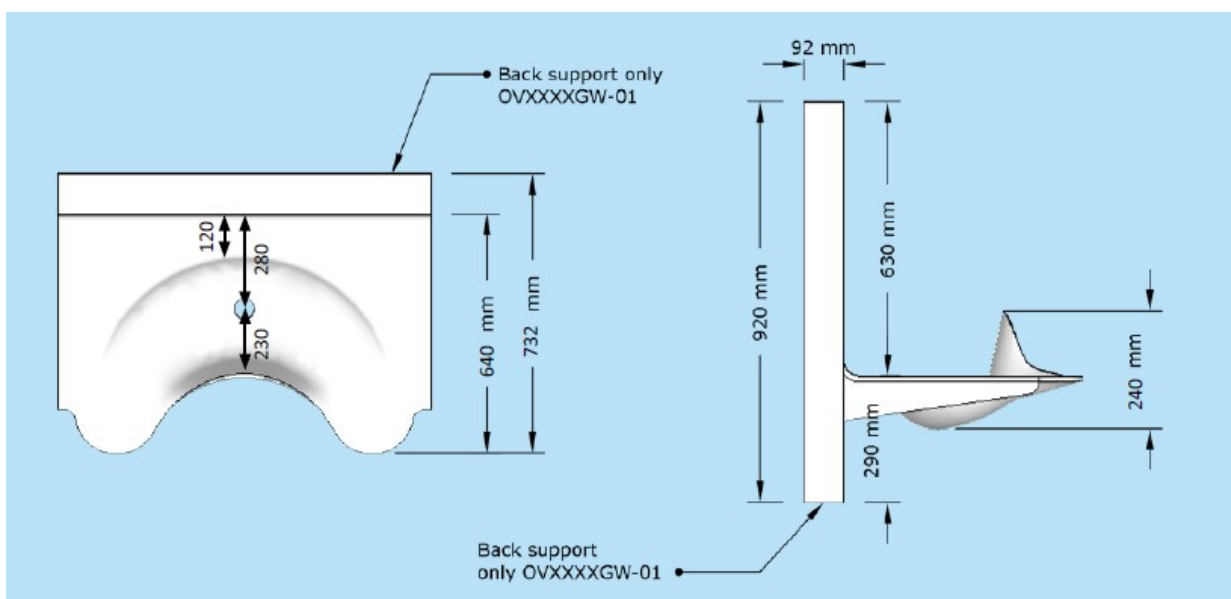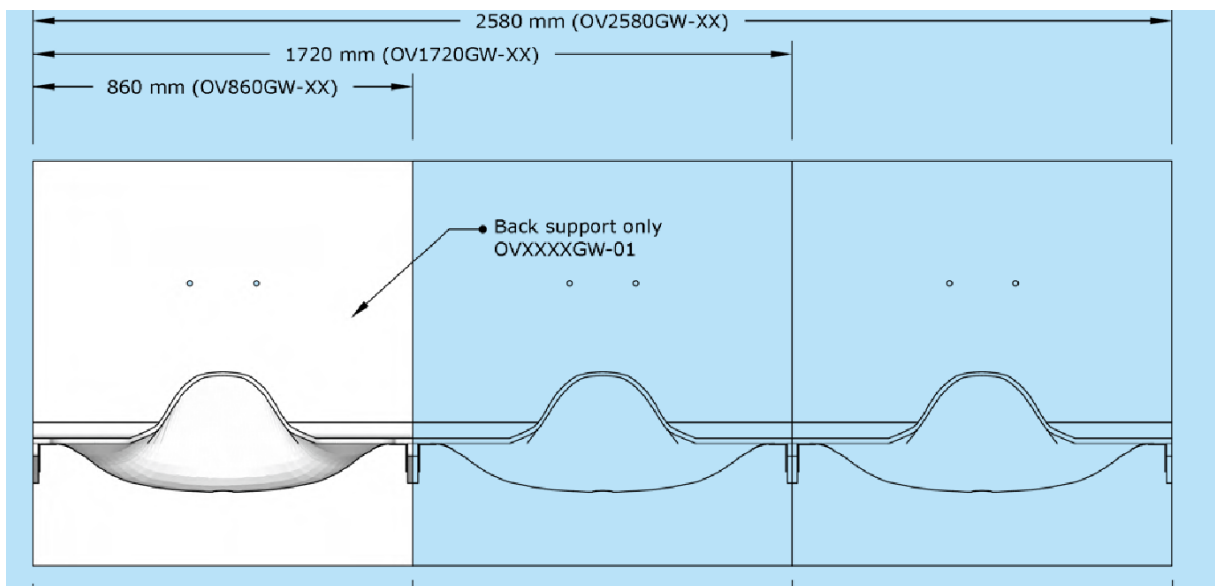

# Surgery Sink - 03

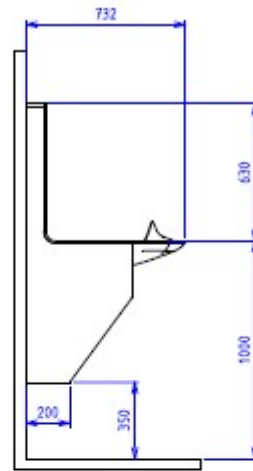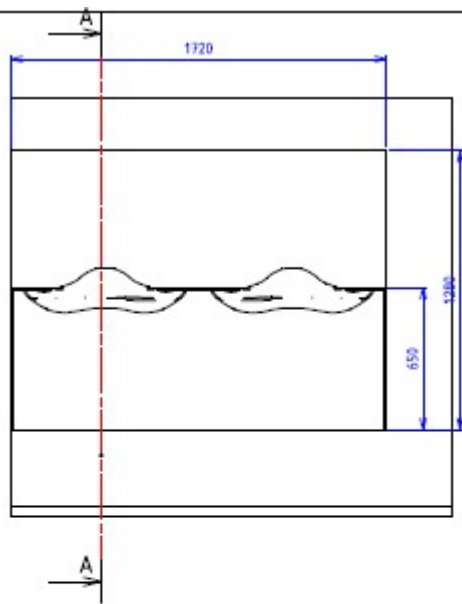

A - A

