

NOAS
SWEDEN®



Installation instructions

Surgery sink OV01



Installation of OV01.

Put a nogging if the wall not is intended for loads. CC between the upper and lower fastener should be 815 mm.


Start from which height the plan of the sink should sit on. Add 578 mm, mark the height on the wall.

Example: Should the sink be 900 mm above the floor, the center hole should be drilled at 1478 mm above the floor.


Unscrew the M6 screws and pull out the bracket from the sink.



Measure out the center sideways and mark. Drill the appropriate hole and hang up the bracket with a screw.

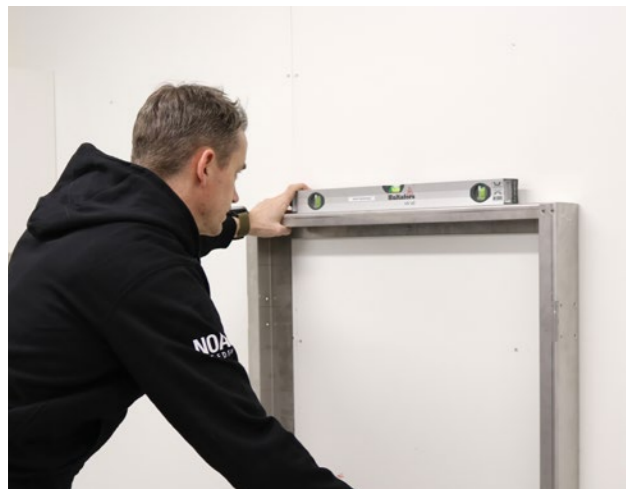

Align the bracket and mark the other five holes.

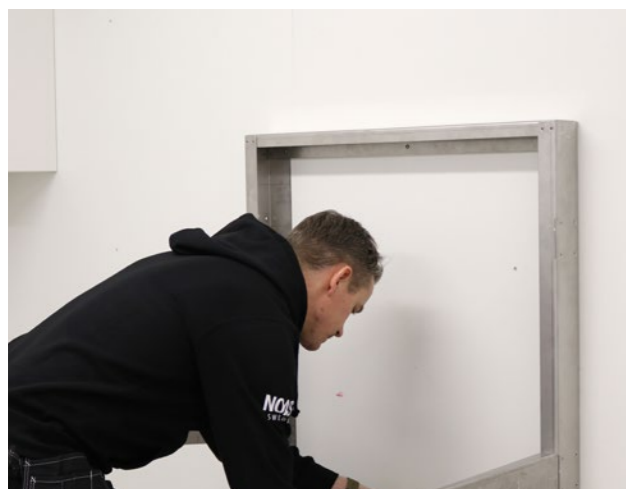

Drill and screw all screws.



Slide the sink onto the brackets.



Make sure that the brackets end up in the slides.



Now, mounting of mixers, drains, dual-dispensers etc. can take place. Remember to always drill holes and use through screws with washer and nut when mounting accessories in the Corian disc. Never screw in a Corian metal or wood screw.

Then push in the sink completely against the wall. Never use force, it should slide easy.



Screw the sink into the bracket with the enclosed M6 screws.

Head office in Tibro

Noas Sweden
Fabriksgatan 2
S-543 50 Tibro
Sweden

Phone: +46 504-49 51 00

info@noassweden.se
www.noassweden.se

NOAS
S W E D E N